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U.S. Department of Health and Human Services

**FDA U.S. FOOD & DRUG ADMINISTRATION**

A to Z Index | Follow FDA | En Español

Search FDA

Home | Food | Drugs | Medical Devices | Radiation-Emitting Products | Vaccines, Blood & Biologics | Animal & Veterinary | Cosmetics | Tobacco Products

**Safety**

Home > Safety > MedWatch The FDA Safety Information and Adverse Event Reporting Program > Safety Information > Safety Alerts for Human Medical Products

**Safety Alerts for Human Medical Products**

2017 Safety Alerts for Human Medical Products

2016 Safety Alerts for Human Medical Products

**Evita V500 and Babylog VN500 Ventilators by Dräger Medical: Class I Recall - Faulty Batteries**

SHARE TWEET LINKEDIN PIN IT EMAIL PRINT

[Posted 09/02/2015]

**AUDIENCE:** Risk Manager, Surgery, Critical Care Medicine, Nursing, Pediatrics

**ISSUE:** The battery (part of the PS500 Power Supply Unit) that powers the Evita V500 and Babylog VN500 Ventilators does not last as long as expected. The battery indicator light shows a sufficiently charged battery even when the battery is depleted. When the "battery low" and "battery depleted" alarms sound, the devices do not indicate how much time is left before the ventilator will shut down due to lack of power. Analysis by Dräger indicates that the battery should last approximately 30 minutes. If the power is lost, the 30-minute battery back-up should last until the ventilator is connected to a main power supply. See the [Recall Notice](#) for a list of affected lot numbers.

**BACKGROUND:** The Evita V500 Ventilator provides constant breathing support for adults and children, including premature babies weighing at least 14 ounces. The Babylog VN500 provides constant breathing support for

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